SONY

LMD-2451MT

24-inch Full HD 3D LCD medical monitor





Overview

3D high definition medical monitor with advanced Sony 3D technology

The LMD-2451MT combines Sony 3D professional technology with an advanced 24-inch LCD panel to display clear 3D pictures.

Surgeons can also view images from multiple monitors whilst wearing light, comfortable polarized glasses.

Practical features include a user memory function and chroma phase control, while ChromaTRU technology contributes to optimised image quality.

This equipment is intended for use by qualified medical professionals only.



Features

Pioneering 3D quality

The LMD-2451MT uses advanced panel micro polarizer filter technology to deliver clear, smooth, consistent pictures that are viewed with passive glasses.

Multiple 3D formats

The versatile LMD-2451MT accepts various sources of 3D signal, such as 3D on 3G, dual stream left and right, field sequential and side-by-side HD-SDI, as well as DVI-D line interleave (line by line) mode. The display can accept almost any signal ranging from SD to HD video, as well as PC signals via its DVI-D or HD15 connectors.

Sony unique colour matching technology ChromaTRU

The LMD-2451MT helps to ensure more accurate colour consistency by means of a series of calibrations. The panel is individually calibrated so that RGB co-ordinates are consistent. A further calibration helps maintain white balance at a uniform colour temperature throughout all grayscale levels.

High brightness and contrast through super-wide WUXGA

The display utilises a wide WUXGA (1920x1200) LCD panel that offers high brightness and contrast.

Natural gradation and accurate colour reproduction

In order to ensure images have a smooth, natural gradation, the LMD-2451MT uses an advanced 10-bit digital video signal processor.

Gamma curve selection

The LMD-2451MT offers users the choice of DICOM or CRT 2.2. as sometimes, the accuracy of images does require a different gamma curve.

Multiple display modes

A choice of different display modes includes picture-out-picture, side-by-side split screen, zooming, and viewing live video all help to view images simultaneously.

Mirror image

The surgeon's assistant can view a mirror image of the surgeon's own display, in order to assist with more convenient manoeuvering of the camera.

Protected controls

In such a sensitive environment, it is vital to prevent inadvertent operation from the control panel. The LMD-2451MT's key-inhibit function helps to remove this risk.

Safety listing and compliance

The LMD-2451MT is fully safety listed and compliant for use in medical environments. The display is UL60601-1 listed and complies with CSA C 22.2 No. 60601 and EN60 601-1 safety regulations.



Technical Specifications

Picture Performance	
• Type	a-Si TFT Active Matrix LCD with an AR-coated protection panel
 Resolution 	1920 x 1200 pixels (WUXGA)
Effective picture size (H x W)	518.4 x 324.0 mm (20 1/2 x 12 7/8 inches)
Effective picture size (diagonal)	613.2 mm (24 1/4 inches)
Aspect	16:10
• Colors	Approx. 16.7 million colors (8-bit)
2D Viewing angle	89°/89°/89° (typical) (up/down/left/right contrast >10:1)
3D Viewing angle	50° at a viewing distance more than 300 mm, crosstalk less than 7% (typical)

Input	
Composite	BNC (x1), 1.0 Vp-p ±3dB sync negative
• Y/C	Mini DIN 4-pin (x1)
•	Y: 1.0 Vp-p ±3dB sync negative,
•	C: 0.286 Vp-p ±3dB (NTSC burst signal level),
•	0.3 Vp-p ±3dB (PAL burst signal level)
RGB, Component	BNC (x3)
•	RGB: 0.7 Vp-p ±3dB (sync on green, 0.3 Vp-p sync negative)
•	Component : 0.7 Vp-p ±3dB (75% chrominance standard color bar signal)
• DVI-D	DVI-D (x1)
•	TMDS single link
• HD15	D-sub 15-pin (x1),
•	RGB: 0.7 Vp-p sync positive (sync on green, 0.3 Vp-p sync negative)
•	Sync : Total level (polarity free, H/V separate sync)
•	Plug & Play function : corresponds to DDC2B
External sync	BNC (x1)
•	0.3 Vp-p to 4.0 Vp-p ±bipolarity ternary or negative polarity binary
Option slot	2 slots
Parallel remote	Modular connector 8-pin (x1) (pin-assignable)
Serial remote	D-sub 9-pin (RS-232C) (x1), RJ-45 modular connector (Ethernet) (x1) (10BASE-T/100BASE-TX)
• DC in	DC 5V / 24V (output impedance 0.05 ohms or less)

Output		
 Composite 	BNC (x1), loop-through, with 75 ohms automatic termination	



• Y/C	Mini DIN 4-pin (x1) loop-through, with 75 ohms automatic termination
RGB, Component	BNC (x3), loop-through, with 75 ohms automatic termination
External sync	BNC (x1), loop-through, with 75 ohms automatic termination

General	
Power requirements	DC IN: 24 V 5.0 A, 5 V 0.030 A (supplied from AC adapter)
•	AC adapter (Sony, AC110MD)
•	AC IN: 100 V to 240 V, 50/60 Hz, 1.53 A to 0.58 A
•	DC OUT: 24 V 5.0 A, 5 V 0.060 A
 Power consumption 	Maximum: approx. 136 W (with 2 x BKM-229X)
Operating temperature	0°C to 35°C (32°F to 95°F)
Recommended	20°C to 30°C (68°F to 86°F)
Operating humidity	30% to 85% (no condensation)
Storage and transport temperature	-20°C to +60°C (-4°F to +140°F)
Storage and transport humidity	0% to 90% (no condensation)
Operating, storage, and transport pressure	700 hPa to 1060 hPa
• Dimensions (W x H x D)	602.4 x 386.2 x 110 mm (23 3/4 x 15 1/4 x 4 3/8 inches) (including projections)
Mass	8.7 kg (19 lb 3 oz) (with 2 x BKM-229X) AC adapter (AC110MD) 1.2 kg (2 lb 10 oz)

Supplied Accessories	Supplied Accessories	
• AC-110MD	AC adapter (1)	
•	AC power cord (1)	
•	AC plug holder (2)	
•	3D glasses (glasses-type) (1)	
•	3D glasses (clip-on type) (1)	
•	L/R labels (1)	
•	Instructions for use (1)	
•	CD-ROM (1)	
•	Using the CD-ROM Manual (1)	
•	Quick Reference (1)	
•	When you First Use the Monitor (1)	
•	Sales Companies Guide (1)	
•	Warranty	

Accessories

Medical Accessories



AC-110MD

AC adapter for medical monitors, working with LMD-3251MT, LMD-3251TC, LMD-2451MC*, PVM-2551MC*, LMD-1951MD and, LMD-2451MD*



BKM-243HSM

HDSDI/4:2:2 SDI Input Adaptor



BKM-250TGM

3G/HD/SD-SDI Input Adaptor



BKM-30GM

3D circular micro-polariser glasses



BKM-31GM

3D circular micro-polariser glasses (clip-on)



CFV-B100

Shield Frame



CFV-E30D

3D Eye Shield



CFV-E30SK

3D Eye Shield Kit



SU-560

Stand specifically designed for medical monitors, working with LMD-1951MD, LMD-2451MD and PVM-2551MD

Option Boards and Cables



BKM-220D

4:2:2 Sdi Input Adaptor



BKM-227W

NTSC / PAL input adaptor for LMD monitors



BKM-229X

Analogue component input adaptor for LMD monitors

SONY



